## MEGA-M LC / LC Flow

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## MEGA-M LC / LC Flow

### Description of the product

MEGA-M LC / MEGA-M LC Flow is a light-cured ready to use material for modelling of Inlays, Onlays, veneers, secondary parts and other modelled constructions and for blocking out undercuts. MEGA-M LC / LC Flow is applicable in syringe (tube), ready to use

### Composition

MEGA-M LC / LC Flow based on multi-functional methacrylates with photo-initiators for blue- and UV-light spectrum. MEGA-M LC / LC Flow burns out free of residue.

### Range of application

- modelling of Inlays, Onlays, veneers, secondary parts, etc.
- blocking out undercuts
- fixation of solder points
- correction and restoration of plaster-models and stumps
- fixation of orthodontics at the model and by metal cast dentures

### Preparation of the model

The model must be free of dust and grease. It must be necessary to insulate the model with an Alginate insulation, if the modelling must be taken out from the model.

### Application

Remove the cap of the MEGA-M LC screw tube, extrude a sufficient quantity of material and recap the tube. MEGA-M LC can be applied with normal instruments by a working time of 3 min. MEGA-M LC Flow can be applied with needles directly at the model.

Apply MEGA-M LC / LC Flow in the desired area to a maximum layer thickness of 3 mm and cure with a UV- or Blue Light Source (e.g. Megalight) for 5 minutes.

MEGA-M LC / LC Flow can be worked out with rotary instruments after polymerization. MEGA-M LC / LC Flow can be polished to high gloss. MEGA-M LC / LC Flow burns out without residue in furnaces by 480 °C / 900 °F.

MEGA-M LC / LC Flow polymerizes under the influence of daylight. Do not extrude paste from the syringe unless you intend to use it immediately.

### Storage

Do not store MEGA-M LC above 25°C / 77°F

### Packaging

MEGA-M LC	Single pack	Syringe 3 g	Ref.: 1M-LC
MEGA-M LC Flow	Single syringe	3 g + 5 needles	Ref.: 1M-LCF
	Assortment	3 x 3 g + 20 needles	Ref.: 1M-LCSY

Date of information: January, 2012